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## Intended use of the product

The ultraSAVER is a medical device.

**Indication:** Suitable for the rescue, transport or evacuation of bedridden or injured persons.

**Contraindication:** Not suitable for the transport of patients with neck or spinal cord injuries.

## User groups

This product may only be used by qualified, expert personnel. A prerequisite is theoretical and practical experience in handling the rescue sheet.

## Important information

Before using the ultraSAVER, please read the instruction for use and safety instructions carefully. Keep the instruction for use in a safe place to avoid operating errors.

## Declaration of conformity

This product was manufactured in accordance with the current EU regulation 2017/745. A declaration of conformity for these products can be acquired from the local dealership or the manufacturer.

## General safety instructions

The ultraSAVER can be used several times. Check the integrity of the rescue sheet **before and after** each use. If damage or only suspicion of damage occurs, take the product out of operation immediately.

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

## Use

The ultraSAVER should only be used by trained personnel. When lifting patients with more than 2 users, make sure that an even tension is built up on the rescue sheet.

## Storage

Store the ultraSAVER in a dry and dark place. Avoid direct sunlight and direct heat radiation from heat sources. These factors can damage the material.

## Maintenance and Care

The rescue sheet can be used several times. Disinfection should be carried out according to the RKI recommendation "surface disinfection after patient contact" or with commercially available disinfectants.

To ensure the safety of the patient and the user, we recommend that the rescue sheet be subjected to **an annual visual inspection** by an expert.

For users in Germany, we recommend observing the following regulations and guidelines: DGUV Principle 305-002 / DGUV Rule 112-198/199 / MPBetreibV.

## Service life

The service life depends on the condition of the product, its frequency of use, and the external operating conditions. The lifespan of the ultraSAVER is dependent on the individual usage conditions. Depending on the frequency and intensity of use of the ultraSAVER, wear and tear is normal. Even with careful handling, the material undergoes an aging process. The product should be disposed of after assessing its condition or no later than 10 years after commissioning. Proper application is assumed here.

---